Gebrauchsanweisung | User Manual

Bracket AED Plus

P/N: 1042100002 (voherige P/N: 2100403032)

■ INHALTSVERZEICHNIS | TABLE OF CONTENTS

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■ WELCOME | SAFETY INSTRUCTIONS

Welcome

Thank you for purchasing the Bracket AED Plus.

In this manual you can find technical information, instructions for use and installation.

Safety Instructions

- Read these instructions carefully before using the Bracket AED Plus.
- Keep these instructions in safe place for future reference and pay particular attention to the safety instructions.
- The manufacturer accepts no liability for damage resulting from improper use, attachment, assembly or installation.
- This bracket is for use only with the ZOLL AED Plus® in combination with the Softbag AED Plus (P/N 1042101002; previous P/N 2104833001).
- Repair and maintenance of the Bracket AED Plus can only be performed by authorized personnel. Please consult directly to CRS medical GmbH Technical Support if the device is defective or damaged. Any changes to the device or its accessories which are not performed by CRS medical GmbH will void all product warranties. CRS medical GmbH will not be liable for unauthorized repair or maintenance.
- Inspect the Bracket AED Plus after every use and do not use if:
 - there is visible mechanical damage on the locking mechanism
 - there is visible mechanical damage on the holder

■ APPLICATION

The Bracket AED Plus is intended to be installed in ambulances which use the ZOLL AED PLUS® only in combination with the Softbag listed under accessories.

The holder may only be used in a protected environment. These include massive walls in the interior of buildings and stable walls of EMS ground vehicles.

Please also follow all manufacturer's instructions for the defibrillator.

■ SCOPE OF DELIVERY

1 Bracket AED Plus (P/N: 1042100002)

1 User Manual

■ TECHNICAL DATA

Weight	3000 g
Dimensions (HxWxD)	260x360x220 mm
Operating temperature	0°C to +40°C, 5% to 95% HR, non-condensing
Storage temperature	-30°C to +70°C, 5% to 95% HR, non-condensing


Subject to change without notice.

■ ACCESSORIES

Softbag AED Plus (P/N: 1042101002)

■ INSTALLATION

Mount the Bracket AED Plus with 4 screws size M6 according to the drilling diagram below.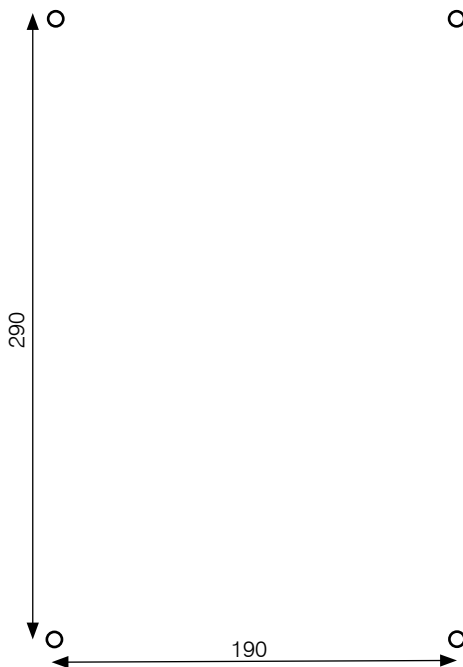


(All dimensions are in mm)

■ OPERATION

Opening

Unlock the bracket by pulling the locking bolt and lifting the locking device.

The locking device rests in the upper position so the device can be removed.

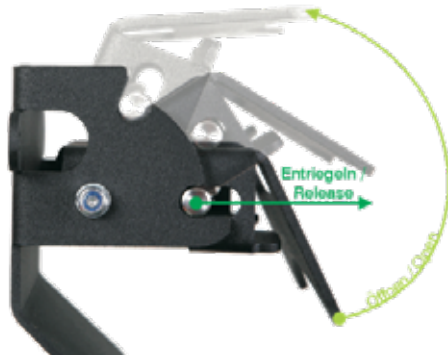Closing

Insert the defibrillator and pull the locking bolt to unlock the locking device from the upper position. Press the locking device down until the locking bolt engages hearable in the lower position.

The defibrillator is now secured.

■ CLEANING | DISINFECTION

Please remove the ZOLL AED Plus® out of the bracket before cleaning.

The Bracket AED Plus can be cleaned with a damp, lint-free cloth and commercially available mild plastic cleaners. Please ensure, that the cleaner does not contain any solvents.

Commercially available disinfectants can be used for disinfection.

■ SYMBOLS

See user manual / note safety instructions

Reference

Lot number

Manufacturer

■ DISPOSAL

Packing is raw material and therefore recyclable. On behalf of environment protection please dispose the packing sorted. Dispose and recycle all packaging according to your local laws and regulations.

For more detailed information on nonpolluting disposal please contact service.

■ SERVICE | MANUFACTURER

CRS medical GmbH
Loherstraße 6
D - 35614 Asslar

T: +49 6441 38331-0
F: +49 6441 38331-29
info@crs-medical.com
www.crs-medical.com

□ Loherstraße 6
□ D - 35614 Asslar

□ T: +49 6441 38331-0
□ F: +49 6441 38331-29

□ info@crs-medical.com
□ www.crs-medical.com